



**A AMK Rehabilitation & HomeCare Supplies**  
**Watercomfort Company**

incorporating

ACN 002 596 072  
ABN 62 002 596 072

Your One Stop Shop - We Come to you!  
Phone (02) **9531 1699** Fax (02) **9531 1799**  
Email: [service@watercomfort.com.au](mailto:service@watercomfort.com.au)

**BATTERY BACKUP POWERLIFT RECLINERS with Dewert motor for CRITICAL CARE**  
**AC06 EA600S1, AC06 EA600S2**

**INSTRUCTION SETUP OF THE DEWERT BATTERY BACKUP UNIT.**

**Note: There is no Audible or Visible observance associated with the changeover from Mains Power to Power being supplied by the above unit. The unit will produce an Audible Alarm when its Battery is nearly depleted and recharge is required.**

The unit is designed to offer continuous operation of the client's power recliner during periods of commercial AC mains failure and is not designed for prolonged periods of discharge. Keep the unit powered to maintain good battery condition and readiness for confident operation

**Set-up Instruction:**

**Connect the 240v mains lead from the AC Adaptor to the mains**

1. The lead must be plugged directly into a 240v 10A GPO - Powered 'ON' PERMANENTLY.
2. This lead cannot be connected through a Power board or other Sharing Device.
3. The Unit is intended for use on Critical rehab equipment.

**Operation Test:**

1. Switch off the Wall Outlet powering the **AC Adaptor to the mains** - test the Critical Equipment whilst the Dewert unit is in circuit.
2. Switch Mains back on ~ The Chair power will return to normal operation.
3. The **AC Adaptor** must be Powered Permanently to maximise standby Power output.

**Do Not:**

1. Disconnect the AC Adaptor from the Mains for any extended period.
2. Open the Dewert unit, as there are no user serviceable parts inside.
3. Allow the Dewert unit to contact or operate in a wet environment.
4. Connect any device other than the Critical Equipment into the Dewert unit.
5. Attempt to clean the unit other than normal dusting and damp cloth methods.
6. Cover the unit in any way to cause loss of Air circulation.

**You May:**

1. Move the Dewert unit with the Power recliner at any time with due consideration to the above.
2. Test the Dewert unit for confidence at your own discretion ~ remember that this device is intended for normal usage and not test exercises – testing once a week is adequate.
3. Clean the Dewert unit with normal electrical safe guards being taken.

**Always return the Unit to Watercomfort for any service or testing requirements.**

**For help with any of the above information – call A AMK Rehab Freecall 1300734664.**